

**CARE AND CLEANING OF
YOUR GEOMATRIX CUSHION**



Blake Medical Distribution Inc.

38 Lincoln Street
Hamilton, Ontario
L8L 7L5


www.blakemedical.ca

CLEANING YOUR GEOMATRIX CUSHION

The cleaning properties of the Geomatrix cushions are unique. All components of the Geomatrix cushion are machine washable. This includes the outer poly-fibre or waterproof covers, the fire-proof cover, as well as the polymer gel layers. Simply disassemble the cushion into the component parts and wash them on the delicate cycle in lukewarm water. The components can then air-dried or dried in the dryer on a delicate cycle. Avoid drying in direct sunlight.


REASSEMBLING YOUR GEOMATRIX CUSHION AFTER CLEANING

The Geomatrix cushions are easily assembled following cleaning. Each cushion cover contains a patch indicating the correct placement and orientation of the gel layers. The Geomatrix Bronze, Silver and Gold/Gold+ cushions are reassembled as follows.

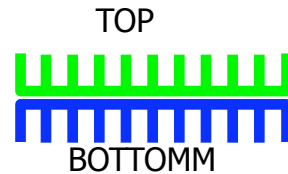


Geomatrix Silver

This three-layer cushion is assembled with the large column blue layer of gel on the bottom with the open end of the columns facing down. The second large layer of the cushion, the green layer, is placed on top of the blue layer with the open end of the columns facing up. The third layer of the cushion, the small blue layer is then placed on top of the green layer with the columns facing up. The layers are then covered with the fire-proof material and inserted into the cover.

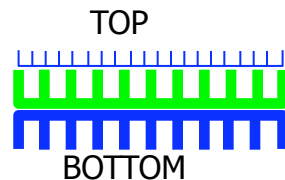

Geomatrix Gold/Gold +

This three-layer cushion is assembled with the large column blue layer of gel on the bottom with the open end of the columns facing down. The second large layer of the cushion, another blue layer, is placed on top of the bottom blue layer with the open end of the columns facing up. The third layer of the cushion, either blue or green, is then placed on top of the green layer with the columns facing up. The layers are then covered with the fire-proof material and inserted into the cover.

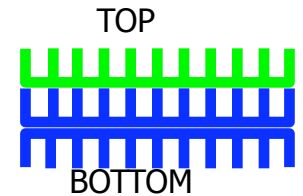

CARE OF YOUR GEOMATRIX CUSHION

Caring for your Geomatrix cushion is simple and straightforward. The cushion is designed to be used at room temperature. If the cushion is subjected to extreme heat or cold, allow the cushion to acclimate to room temperature before using.

The gel layers are light sensitive and will begin to breakdown if exposed for prolonged periods to direct sunlight. Avoid air-drying outside and exposure of the gel layers to direct sunlight.

The gel layers of the cushion are designed to be covered with a fire-proof cover. To ensure that the cushion retains its fire-retardant characteristics, ensure that during reassembly the fire-proof cover is used.